

Model Pro
Series

 **DIPLOMAT
DENTAL
SOLUTIONS**

**The first smart
dental units.**



Model Pro 500.



Model Pro 700.
(with 6 instruments)



Model Pro 600.



Model Pro 800.
(with 6 instruments)



Enter the era of smart dentistry.

The world is rapidly moving towards digitalization, and dentistry is no exception. Electronic health records, digital imaging, and AI-driven technologies have already transformed the dental landscape.

Smart dental units are the logical next step in this evolution. Intuitive tablet controls provide dentists with seamless access to patient records, treatment plans, and educational materials. Enhanced patient education tools enable dentists to communicate complex procedures more effectively, fostering a deeper understanding and trust with patients.



Smart, smarter,



Model Pro.

Smart in communication.

Meet the first dental unit controlled by a mobile app on your tablet. Interactive software empowers patients to be active participants in their treatment decisions. Instill confidence and build lasting relationships.

Smart in ergonomics.

Thoughtful design and adjustability options offer you optimal comfort during procedures, reducing strain and fatigue. Maintain peak performance and deliver top-quality care.

Smart in daily dental operations.

Smart dental units streamline daily operations, from patient scheduling and billing to inventory management. Digitize your workflow and free up your valuable time.




Smart in communication.

Impress your patients with your newfound digital expertise. Present their treatment plan or educate them about hygiene or prevention using state-of-the-art tools. Apps, images, slides, or videos will support you to showcase your competence and confidence.



Patient education.

Educational videos, software, or presentation are always at your side. Teach your patients about oral care in an easy and interactive way.

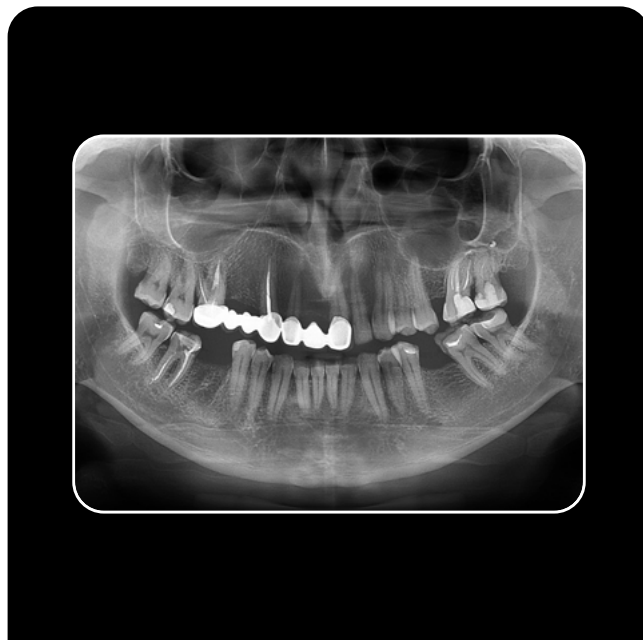

Imaging software integration.


Pull your scans out on the tablet screen and consult with your patients. In a matter of few taps.





Patient management.

No need to run from the dental unit to your desk just to check the treatment history. You can use your patient management software directly on the tablet.

Smart workflow of dental unit

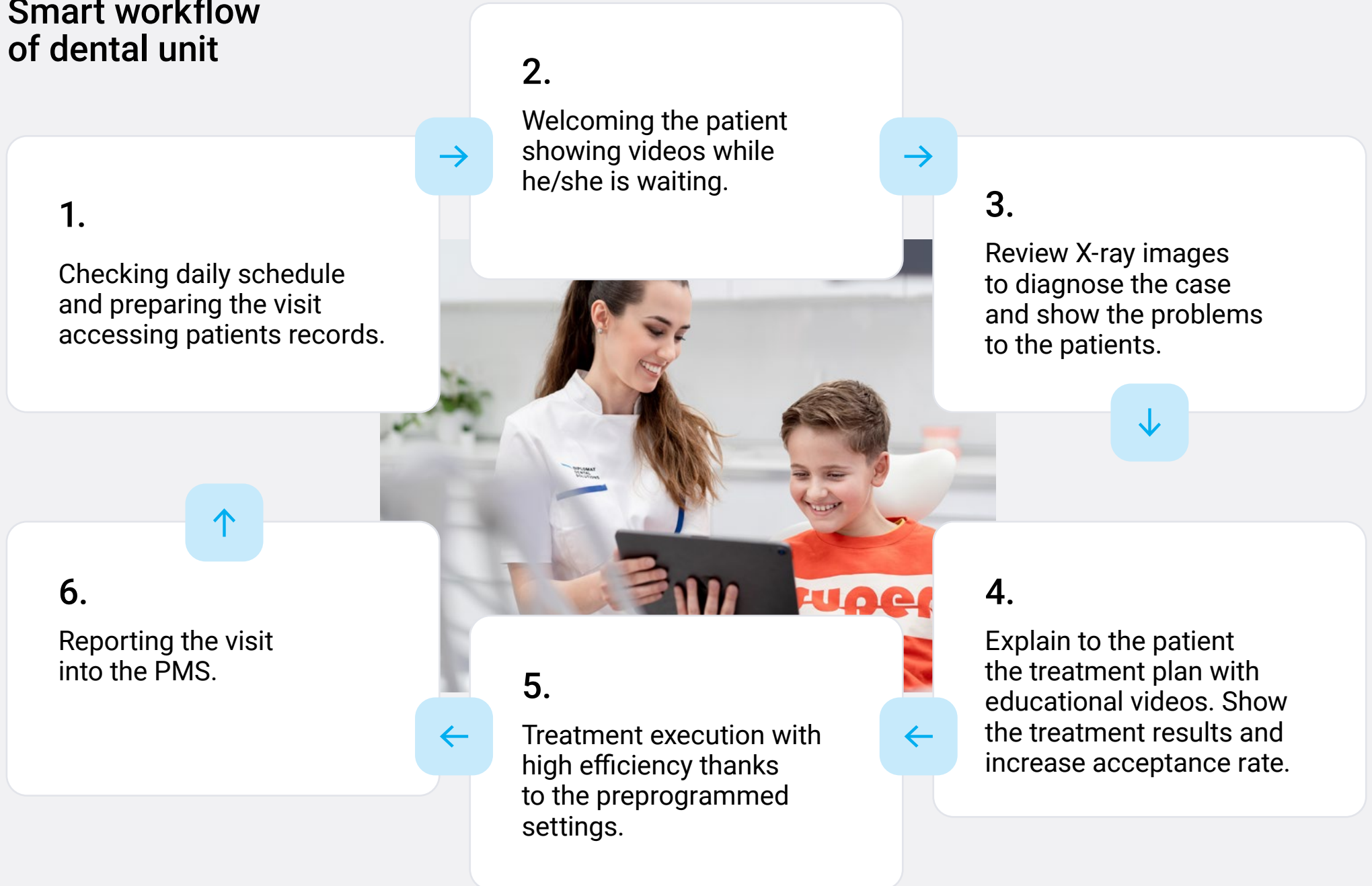

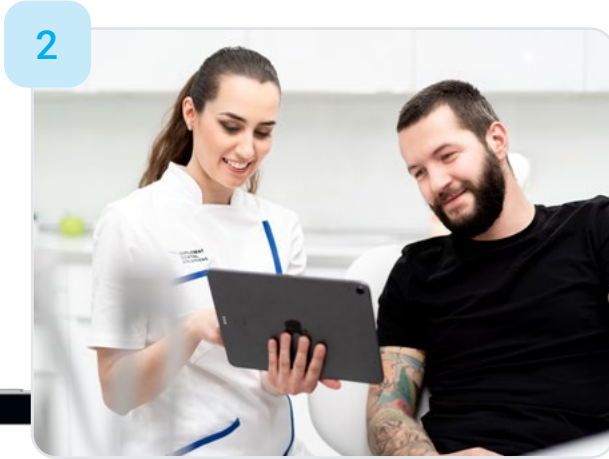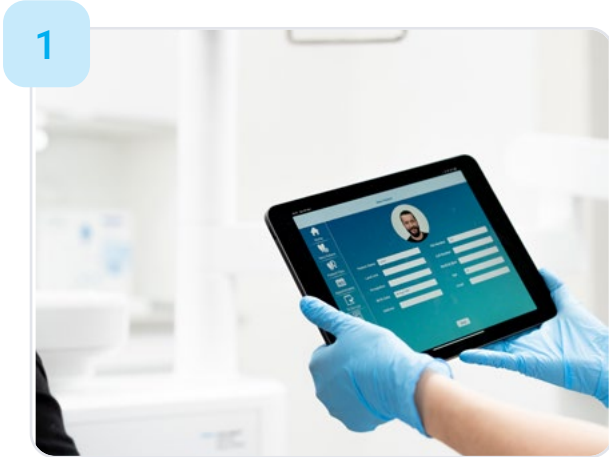




Smart in ergonomics.


The majority of dental staff suffer from back pain and carpal tunnel syndrome. Our design philosophy is based on the belief, that your daily work should never put a heavy toll on your body. We also apply the same principles to patients. Lead a healthy and balanced work life with our smart ergonomic solutions.




Some of the ergonomic highlights:

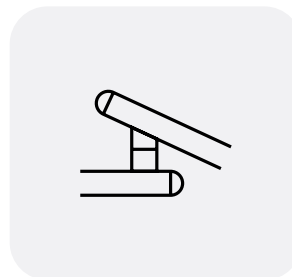

The dental unit's arms are ergonomically adapted for your daily work.

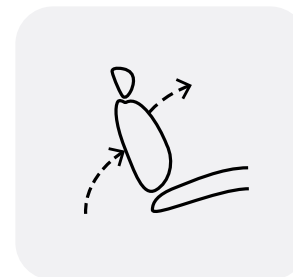

The backrest allows the sideways movement of the patient's upper body.

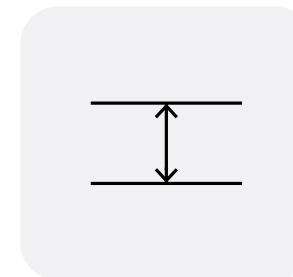


Minimum backrest thickness to allow dentists to get close to the patient's head.

Model Pro takes care of you.

It's not just your patients but you that deserve the best. Musculoskeletal problems appear to be the leading cause of dental practitioners' premature retirement from practice.

Great ergonomics are essential for your long-term health. Model Pro meets all the conditions for ergonomic work, from palatine treatment to operational positions.

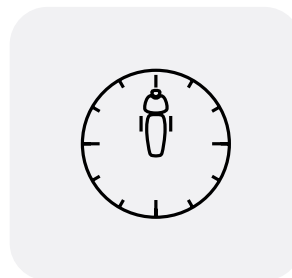

The dentist's and assistant's elements can get incredibly close for a two-handed treatment at the 12 o'clock position.





The assistant's element can free the space for four-handed treatment.

Up or down? Up to you!

Continental style.



Traditional style.

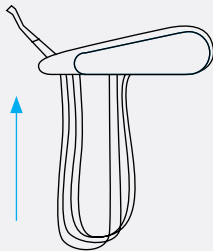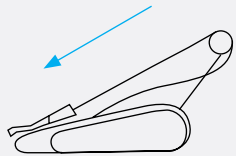


Continental Style

Upper hose delivery system.


Traditional Style

Lower hose delivery system.

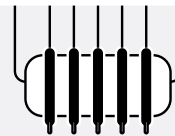

5 instruments table is by default on:


Model Pro 500 & 600

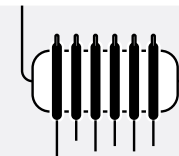

6 instruments table is by default on:


Model Pro 700 & 800

Preserve your health.

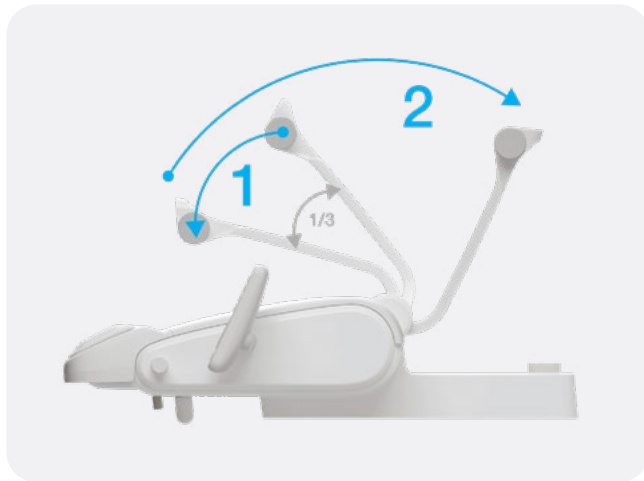

No pull-back.

Completely new construction of whips and re-balancing of instruments places no strain on your arms and wrists whatsoever there is no pulling back by your whips. The whip is pulled towards the stop to unlock. The whip then returns to its resting position.

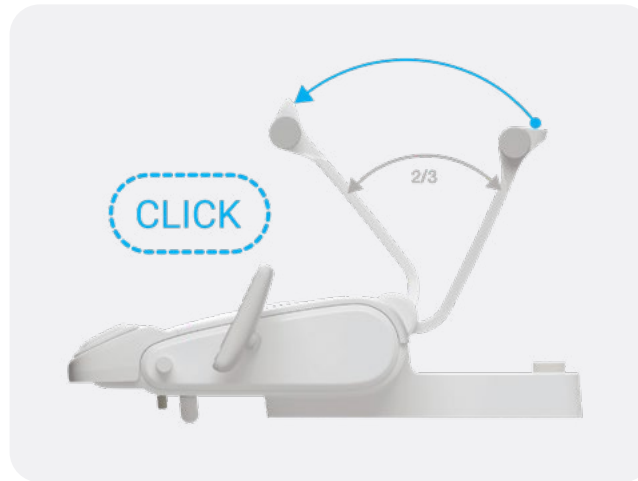

Smooth and easy movement.

Thanks to the pneumatic break in the pantographic arms and rolling bearings on rotary components.



Hold the whip hand.

The dentist's element with instrument hoses routed in the upper hoses configuration may be equipped with a whip lock an instrument whip is routed to the instrument placed in a designated slot

Love at first seat.

Model Pro's design follows ergonomic principles to ensure comfortable experience for you and your patients.

- a) The backrest is wide enough to ensure patients' upper body side movement.
- b) For the most demanding dentist, a 3D headrest is essential to allow a lateral flexion position of the patient head.


Headrest accepts and copies the shape of the neck and head of any patient.


Shoulders naturally drop so as to ensure a comfortable back position.

Seat and backrest relieve the patient and make them feel supported.

The Trendelenburg position in a standard feature.

Lying position with legs fully supported and arms resting comfortably to enhance relaxation.

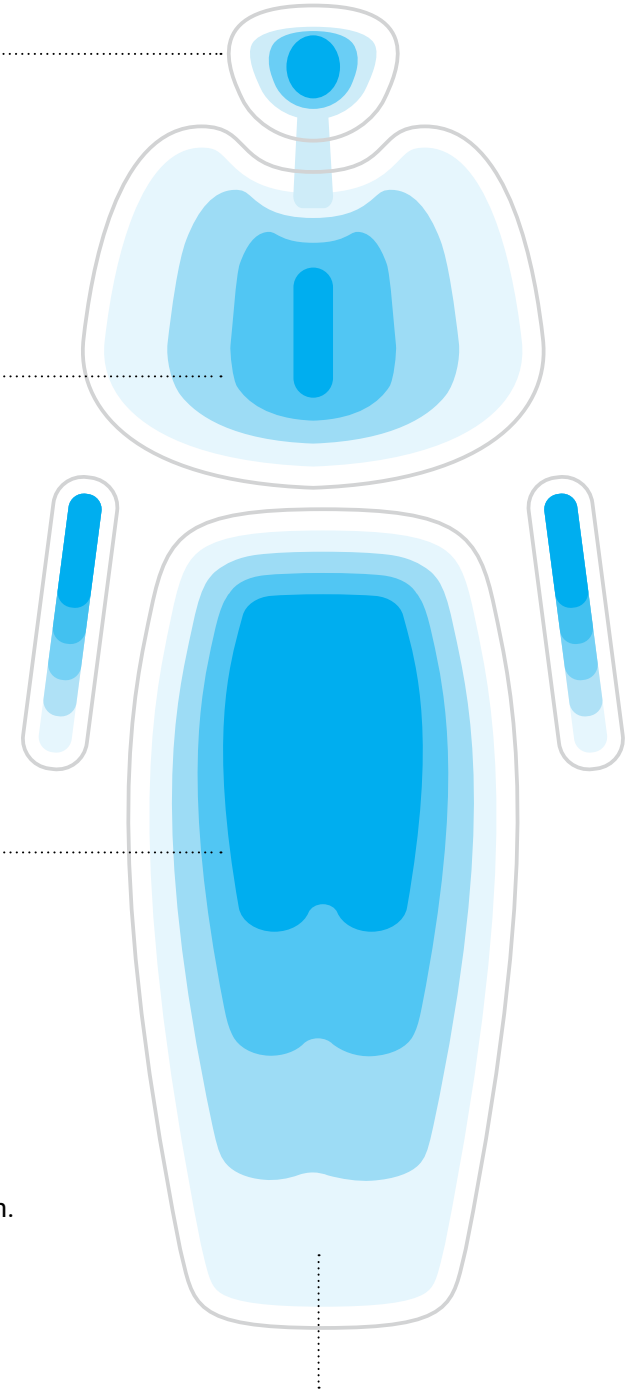

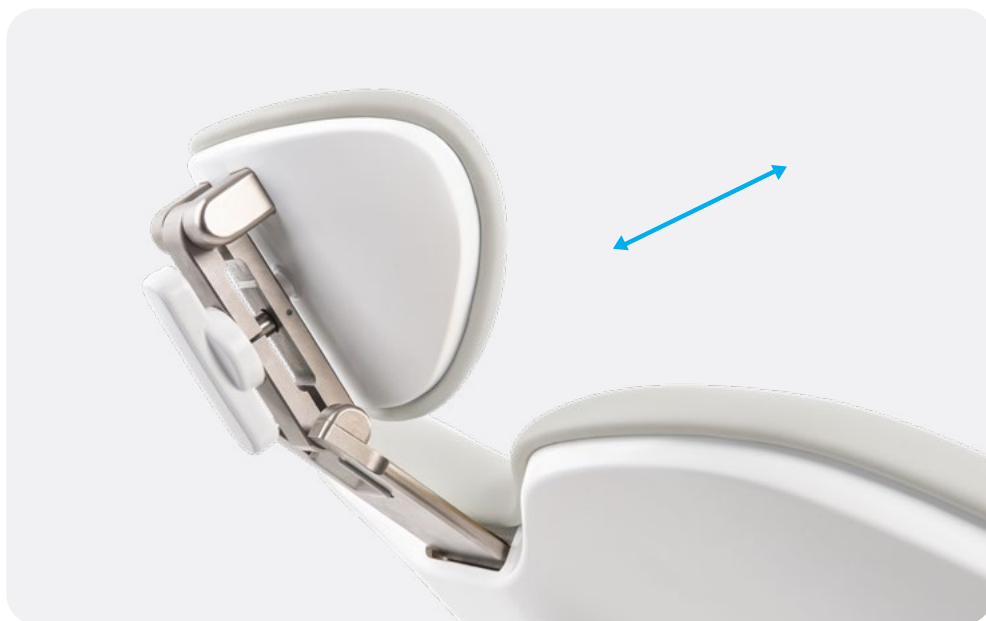

2D Mechanical headrest

Standard
equipment

The mechanical headrest incline is set by releasing the lever located on the back of the headrest. After reaching the desired position, the back-rest must be locked again with the lever.



3D Pneumatic headrest

Optional
equipment

The pneumatic back-rest incline is set by pressing the pneumatic brake button. When the desired position is reached, release the pneumatic brake button.

Height setting is performed mechanically, by pulling or pressure in the direction of the setting.

Folding knee chair for Model Pro 600/800.



Designed with accessibility in mind.

Easier access for patients with mobility problems. Innovative seat design provides a seamless and effortless experience for patients when sitting down and standing up, minimizing physical strain.

Combined with the tablet solution, Diplomat Model Pro provides one of the best dental chairs for patient communication, enhancing both engagement and overall patient experience.

An ideal solution for public healthcare institutions and clinics treating patients with disabilities, ensuring every patient feels at ease in the dental chair.



**Automatically
extending
footrest.**

Paired with the lift version of Model Pro, which lowers the seat closer to the ground (350 mm), this feature significantly enhances accessibility – especially for patients with mobility challenges or those requiring extra support.

Facilitates more natural patient communication while keeping the patient comfortably seated for treatment discussions.



Foot controller support.

Improve your posture and ergonomics during work hours using compatible foot controllers.

You can choose both wired and wireless options wireless foot controller has an average battery lifespan of 2 to 3 months the recharging process takes 24 hours use your your dental unit without limitations.



NOK

Configurable via Diplomat Connect App.



SWING

Horizontally operated multifunction foot pedal for controlling instruments, chair, and water unit. With Improved ergonomics and easy control of the instruments' spray with the lever.

An extra arm comes in handy.

Assistant's element comes with either basic or pantographic arm. Upgrade to pantographic arm to give your team even more workspace.



And yet it moves!

Model Pro's spittoon bowl comes in two versions. The rotary bowl can be either mechanical or electronic.



What light will you turn on?

You can choose from the XENOS or the MAIA dental lamps. The Xenos lamp reaches up to 26.000 Lux. The MAIA lamp up to 35.000 Lux and CRI>95. Both lamps have hands-free operation functions thanks to the no-touch sensor.



Available in options

SIRIUS Essential

x4 LED | 35.000 Lux | 5500 K LED temperature | No composite mode | Sharp illumination area | Smooth adjustment of illumination intensity across the full min/max range

SIRIUS Plus

x8 LED | 35.000 Lux | LED temperature 5500 K | Composite mode ("Yellow" – light with filtered blue light wavelength 430–480 nm) | Sharp illumination area | Smooth adjustment of illumination intensity across the full min/max range

SIRIUS Ultra

x8 LED | 50.000 Lux | x4 different LED temperatures: Warmer white light – 4000 K, Warm white light – 4400 K, Neutral white light – 4800 K, Cold white light – 5500 K | Composite mode ("Yellow" – light with filtered blue light wavelength 430–480 nm) | Sharp illumination area | Smooth adjustment of illumination intensity across the full min/max range (at LED temperature of 5500 K) | Four steps adjustment of illumination intensity across the full min/max range (at LED temperature 4000–4800 K)





Smart in daily dental operations.


The future in now. Digitize your daily operations with Model Pro and its smart features that will not only improve your daily workflow but will also amaze your patients.



Diplomat Connect app.

Imagine having an extra team member whose task is to memorize and set up even the tiniest details to make your workday as comfortable as possible; without the payroll. That is exactly what the Diplomat Connect app does.

The Diplomat Connect app is integrated into the Model Pro and helps **control and automate the dental unit** and smart instruments with just a few taps. The application runs on a companion tablet device and is connected with the dental unit via Bluetooth.

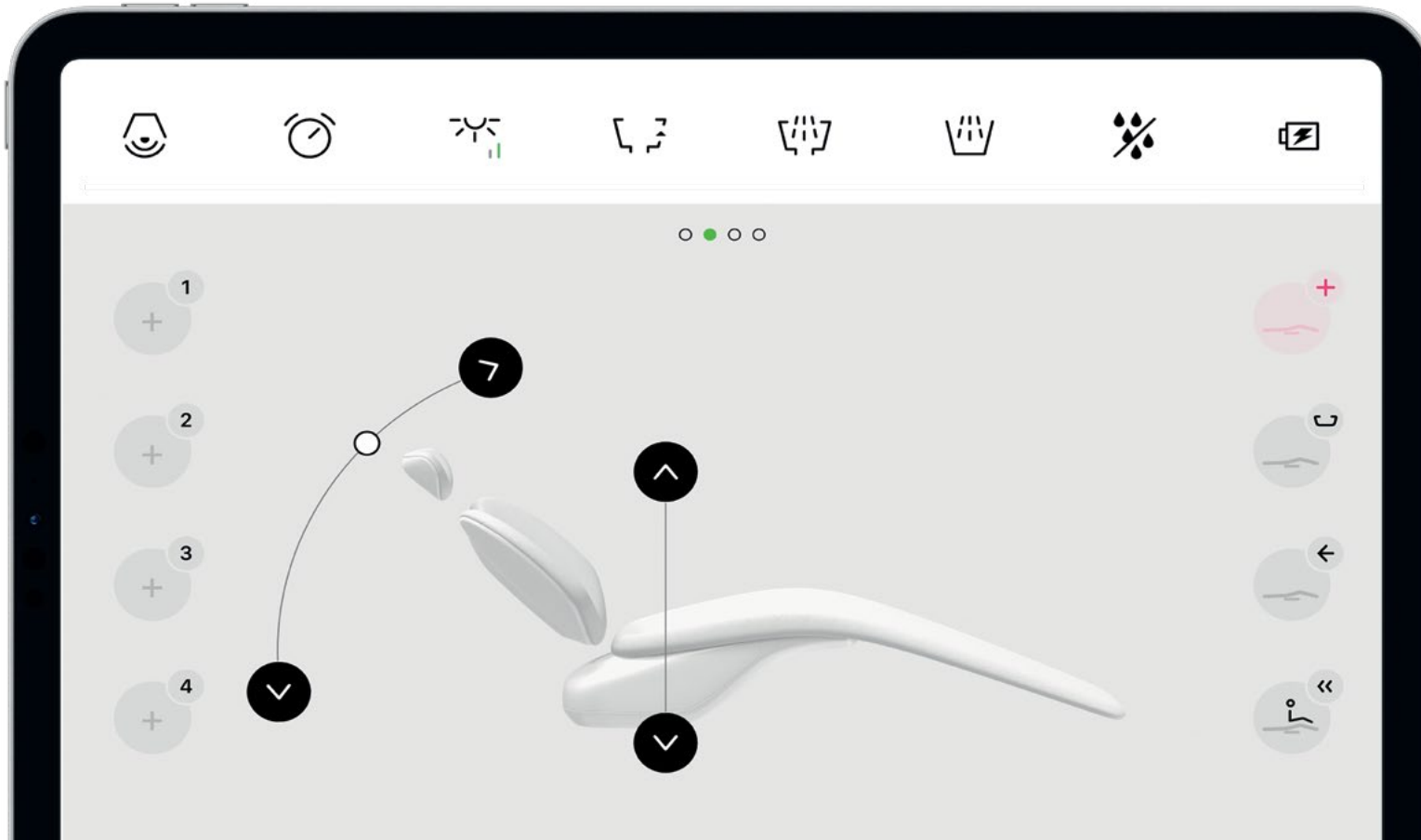

Operate the unit.


Operate the unit. In a easy, intuitive, yet modern way. Save your desired workflow for even better performances.

Control dental unit's instruments.


The user interface will give you quick access to all instruments settings.

Flexible and Open solution.


Use the tablet as unique source to access patient's data, x-ray images, photos and increase the treatment acceptance rate.

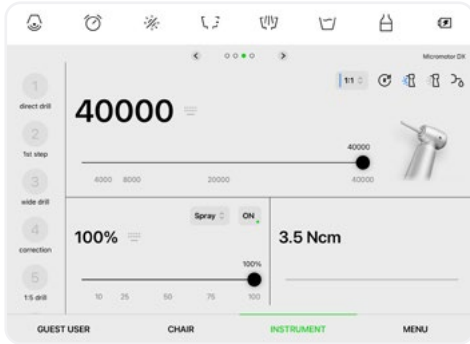


Micromotor.

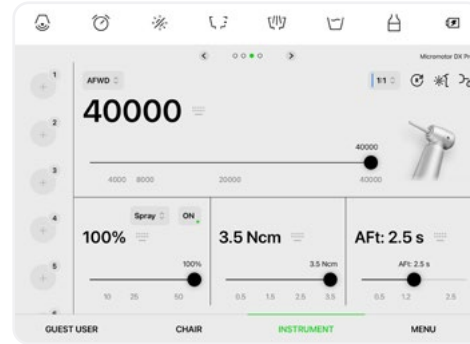

Integrated ENDO function.

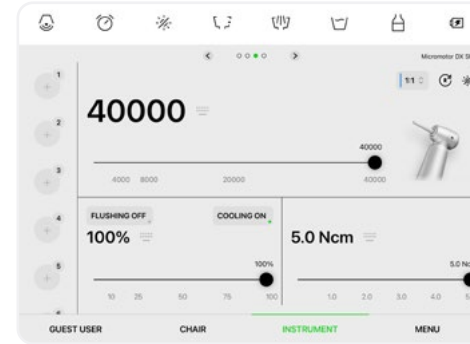

Intergrated Implant function.

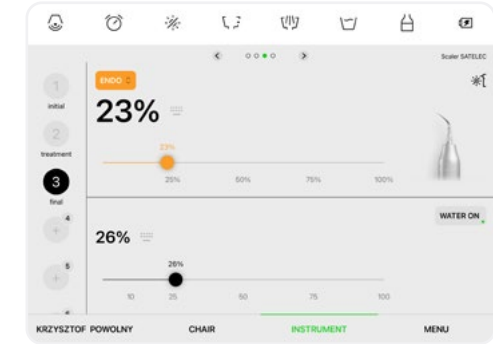




Scaler settings.

Pick your scaler.

The selection of scalers compatible with Model Pro includes:

Scaler	PERIO	ENDO	SCALING & GENERAL	BOOST
Satelec SP Xinetiс	●	●	●	
Satelec NEWTRON	●	●	●	●
Satelec NEWTRON LED	●	●	●	●
NSK VA170	●	●	●	
NSK VA170 LED	●	●	●	
WOODPECKER		●	●	

EMS The EMS scalers do not have specific functions. Their intensity range in from 0–10, which covers all modes.

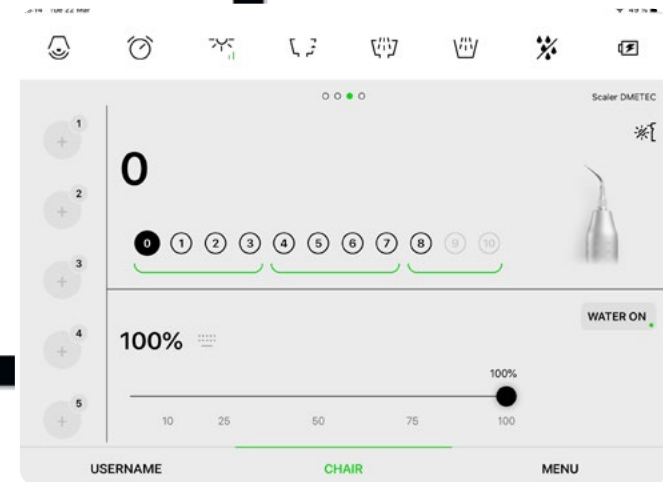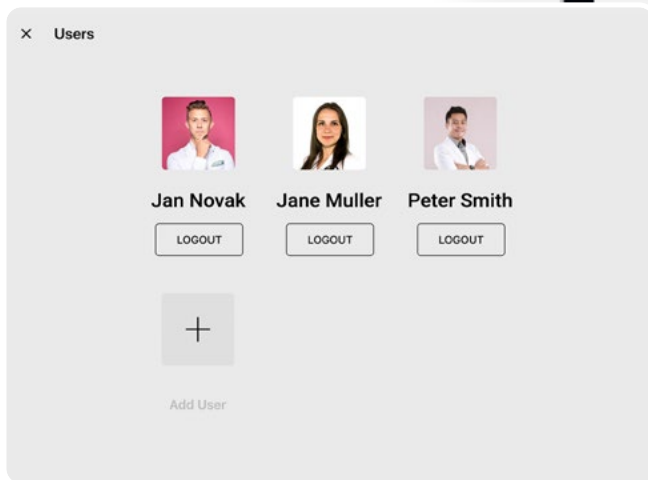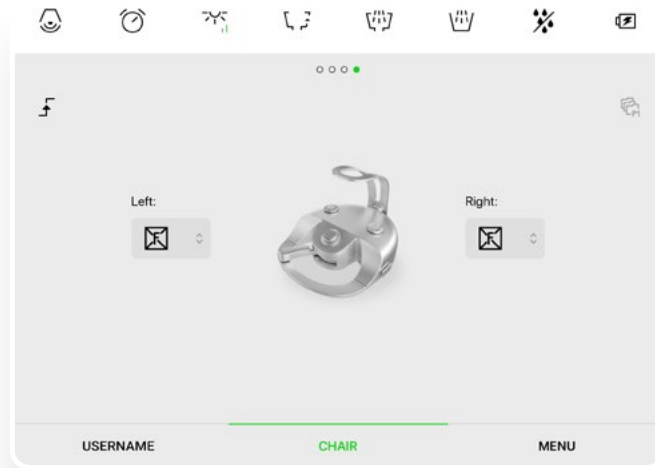

Choose your player.

Everyone gets a unique and personalized experience using the Model Pro. Setup your dental unit's chair positions, instrument parameters, light, hygiene, and much more. You can create an unlimited amount of presets and accounts to suit all your team's needs.

Like clockwork.


Diplomat Connect app does the basic diagnostics automatically. It can analyze and send information on errors to our technicians and keeps your dental unit's firmware always up-to-date. This significantly decreases the downtime of your Model Pro.

We have your back.

We understand that accidents do happen. But do not worry, in case your tablet device breaks, all your presets are safely stored online. You only have to set it up once.




Save your time.

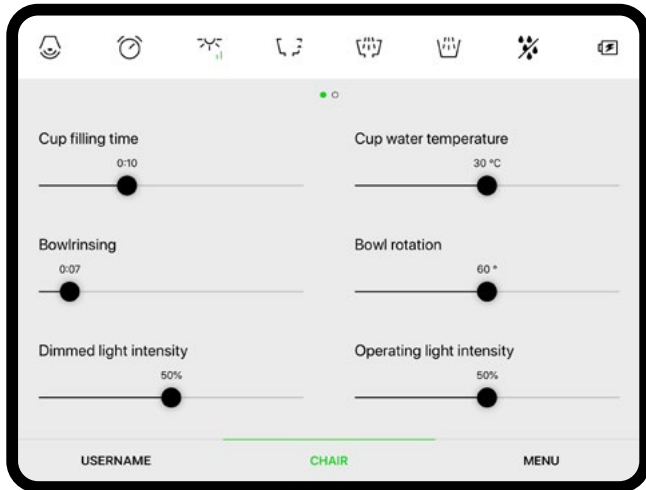

Customizable treatment presets.

Pre-program chair positions, instrument settings, and lighting preferences for common procedures. This feature minimizes separation time and ensures a consistent and optimized workflow for different treatments.

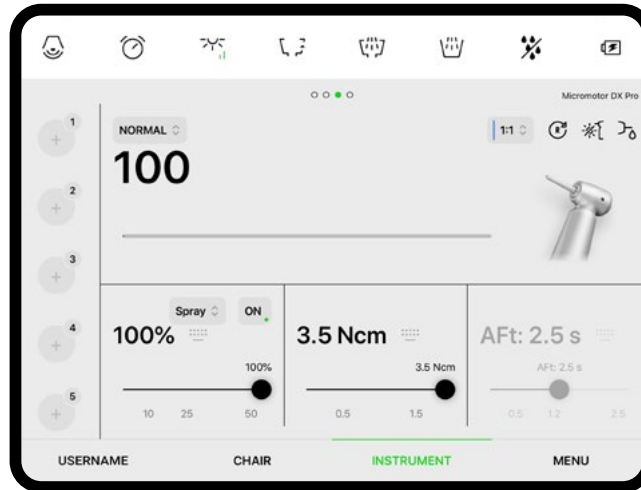

Automated instrument switching.

Transition seamlessly between different instruments and call the preset settings with a simple command, streamlining the treatment process and saving precious chair time.



Integrated digital imaging.

You can now efficiently capture intraoral images or X-rays without leaving the patient's side, eliminating the need to move between different devices or rooms.

Easy convertibility.

Convert your dental unit from left to right just in 30 seconds. Lift the seat, release the brake, rotate the unit, and re-apply the brake. Just like that, faster than a coffee break. The features apply to the Model Pro 500 and the Model Pro 700.



Lift the seat.

Release the brake and rotate.

Switch from
left to right in
30 sec



Diplomat Dental by Bien-Air – Swiss precision at your fingertips.

Your handpiece is more than just a tool – it's an extension of your expertise.

That's why **Diplomat Dental handpieces by Bien-Air** are designed to deliver unmatched precision, durability, and comfort for every procedure.



Uncompromising quality.

Swiss-made excellence ensures smooth operation, perfect balance, and long-lasting performance.

Exceptional reliability.

Built to withstand intensive daily use while maintaining consistent power and minimal vibrations.

Peace of mind guaranteed.

Enjoy a 2-year warranty, ensuring long-term reliability and confidence in every procedure.

Global technical support.

Wherever you are, expert service is always within reach.

Seamless integration.

Elevate your Diplomat Dental unit with high-end Swiss instruments for a superior clinical experience.



Pro line.


- Diplomat CA NOVA 1:5L
- Diplomat CA NOVA 1:1L
- Diplomat TORNADO LED/LK

Classic line.

- Diplomat CA 1:5L
- Diplomat CA 1:1L
- Diplomat BORA 2 LED/LK
- CA 20:1L Micro series
- PM 1:1 Micro series

Precision, durability, and performance — because great dentistry starts with great instruments.

Smart in everyday hygiene.

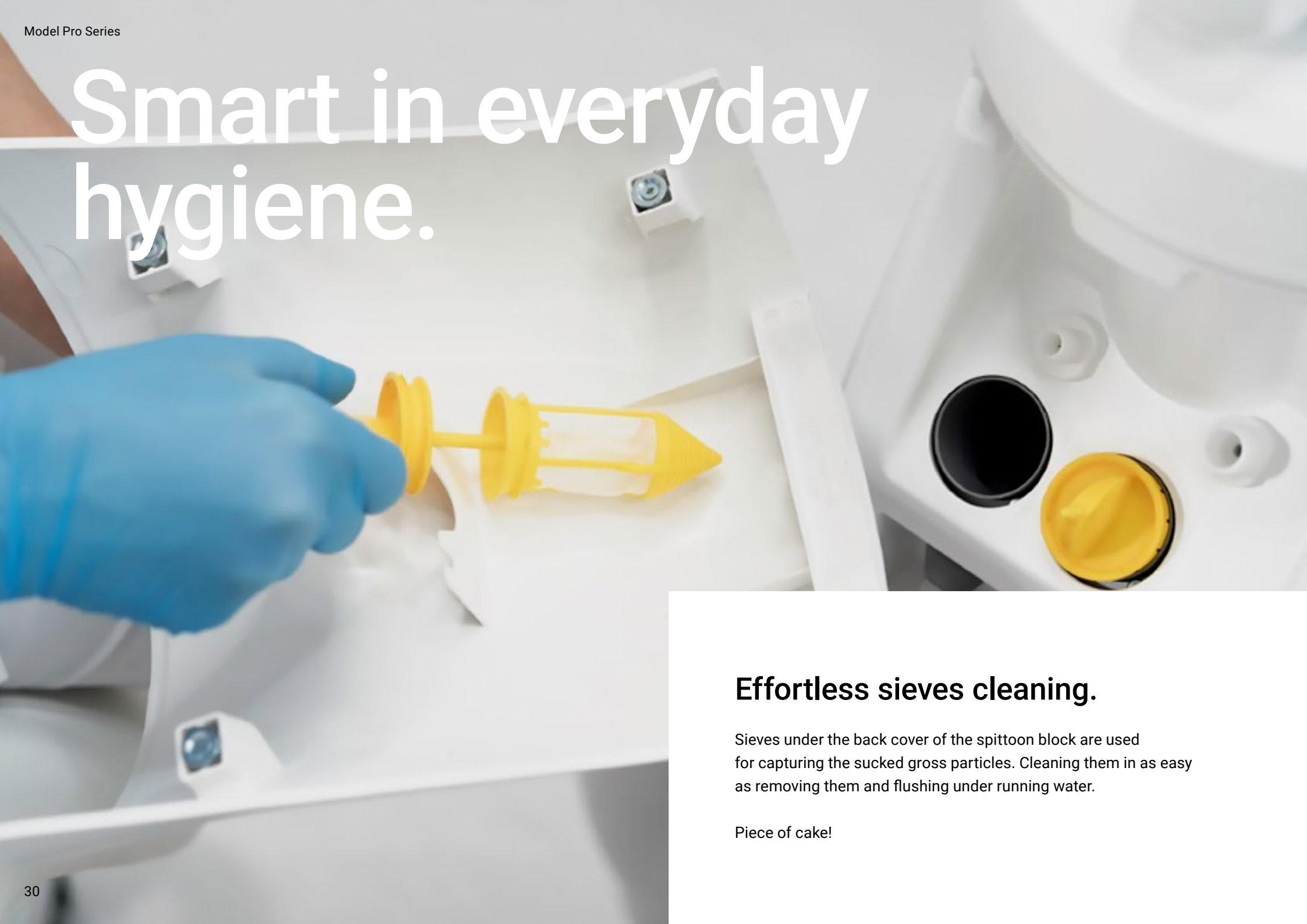

Effortless sieves cleaning.

Sieves under the back cover of the spittoon block are used for capturing the sucked gross particles. Cleaning them is as easy as removing them and flushing under running water.

Piece of cake!



Hygiene holder.



Instrument hoses in hygiene holder.



Automatic hygiene of big and small suction hoses.

Intensive disinfection of instrument water channels.

The recommended disinfectants are Alpron, Sanosil S003, or Dentosept P in a 100% concentration.

The cooling system channels are kept clean and there is no need to use other means of disinfection*.

Disinfection of the instrument cooling waterways in possible to have:

- 01) Manual
- 02) Semi automatic
- 03) Automatic

Automatic hygiene has 4 settings options:

- 01) Beginning of a workday hygiene.
- 02) After treatment hygiene.
- 03) End of a workday hygiene.
- 04) Intensive hygiene.





Smart design.

The Model Pro in a design collaboration by ergonomic researchers, engineers, and technology innovators to level up the experience with dental units as we know it.

Designed to impress.

Smart design makes your work more beautiful and fascinating. It gives you joy simply because it's beautiful.

It also communicates your expertise and uniqueness to your patients. It lets your patients know, they've come to the right place to solve their problems.

Model Pro design won multiple awards at the most prestigious products design competitions all over the world.



reddot design award



BIGSEE
PRODUCT DESIGN
AWARD 2021
WINNER

S L O V A K
D E S I G N
A W A R D

2021
Product Design



DESIGN
AWARD
2022

The Model Pro your way.

Customize the chair's colour selection and upholstery according to your taste. With dozens of options to choose from, it is time for your personality to shine.

No-sew upholstery.

It is never too late to become a hobby interior designer. Choose from up to 49 different colours to complete the interior of your dental office. Your dental office, your decision.

Sewn upholstery with memory foam.

With the sewn upholstery option we have added 20 premium colours to choose from. The memory foam in the chair is used to further increase the comfort for your clients.

Standard Collection.

By picking the Standard version, you will be able to enjoy the typical leather look. This soft-to-touch and flexible material is highly durable, resistant to scratches, and well-protected from UV radiation.



Special Collection.

By picking the Special version, you will be able to enjoy the typical leather look. This soft-to-touch and flexible material is highly durable, resistant to scratches, and well-protected from UV radiation.



Special Plus Collection.

Light material with high flexibility and a perfect finish. By being phthalate-free, the Swiss-made Special Plus version brings extra safety for the user while being easily cleanable and meeting all fire safety standards.



Premium Collection.

With its natural grainy look, the Ultra-leather Premium version delivers a quintessentially classic style that elevates the experience of leather. It is soft to the touch with reinforced rayon backing and industry-leading cutting-edge durability. Premium stitched upholstery is available to order only with active foam to ensure maximum patient comfort.





**Find the details in our
Upholstery Catalogue.**



Smart in diagnostics.

The combination of Model Pro and Model X offers the perfect diagnostic solution for dentists seeking an intuitive, user-friendly, and efficient system for everyday practice. Designed for seamless integration, this pairing ensures effortless operation, enhancing workflow efficiency while maintaining precision and reliability. With advanced features tailored for modern dentistry, Model Pro + Model X provides a streamlined, hassle-free diagnostic experience, making it an ideal choice for professionals who value both performance and simplicity.



Model X 3D 9x14
PAN + CBCT
PAN + CBCT + CEPH

Model X 3D 21x16
PAN + CBCT
PAN + CBCT + CEPH

Model X 3D 0,2 FS (9x14)
CBCT

Model X. The simplest digital diagnostic, that's it.

Diplomat offers the most seamless integration between Diplomat Dental chairs and CBCT. It makes diagnosis easier and enhances the patient experience like never before.

Diplomat Model X offers a combination of features, integrated with AI algorithm, which provide superior image quality that makes diagnosis a piece of cake.

Diplomat Connect X imaging software not only enhances your treatment planning but also simplifies patient communication, ensuring clearer explanations and greater trust.

Diplomat digital workflow.

Transform your practice in 5 steps.



1

Embrace Smart Dentistry with Model Pro.

Transform your practice with the digital and smart Model Pro chair – the smart dental unit designed to enhance patient communication and deliver the best possible patient experience, streamline workflow and simplifies care.

2

Showcase. Streamline. Succeed.

Model Pro tablet allows you to present and visualize the treatment plan. Clear communication builds trust and boosts case acceptance. Instill confidence and trust in your patients, visualizing their oral health.

With the Diplomat Connect App you can save your protocols and streamline your workflow for maximum efficiency, making your treatments more profitable.

3 Simplify diagnostics with Model X.

Adopt Diplomat Model X 3D for the simplest and effortless diagnostics. With simple workflow, superior image quality, and simplified treatment planning, Model X makes your diagnosis easy.

4 Integrate Model Pro and Model X.

Enjoy effortless integration between Model Pro and Model X to accelerate treatment execution. Get X-ray images directly at the chair, while sitting next to the patients. Show them your treatment plan. Instill confidence and trust in your patients.


5 Future-proof your dental practice.

Transform your practice with Model Pro and Model X – moving from analog to digital you're not just upgrading equipment. You're delivering a better patient experience and you will have a more profitable workflow.



Smart in business.

Become an early adopter of this groundbreaking technology. Being the first in your region or community to offer a smart dental unit can position you as industry leader going forward. Model Pro Series will not only greatly impact your daily work-life, but even the its business side.

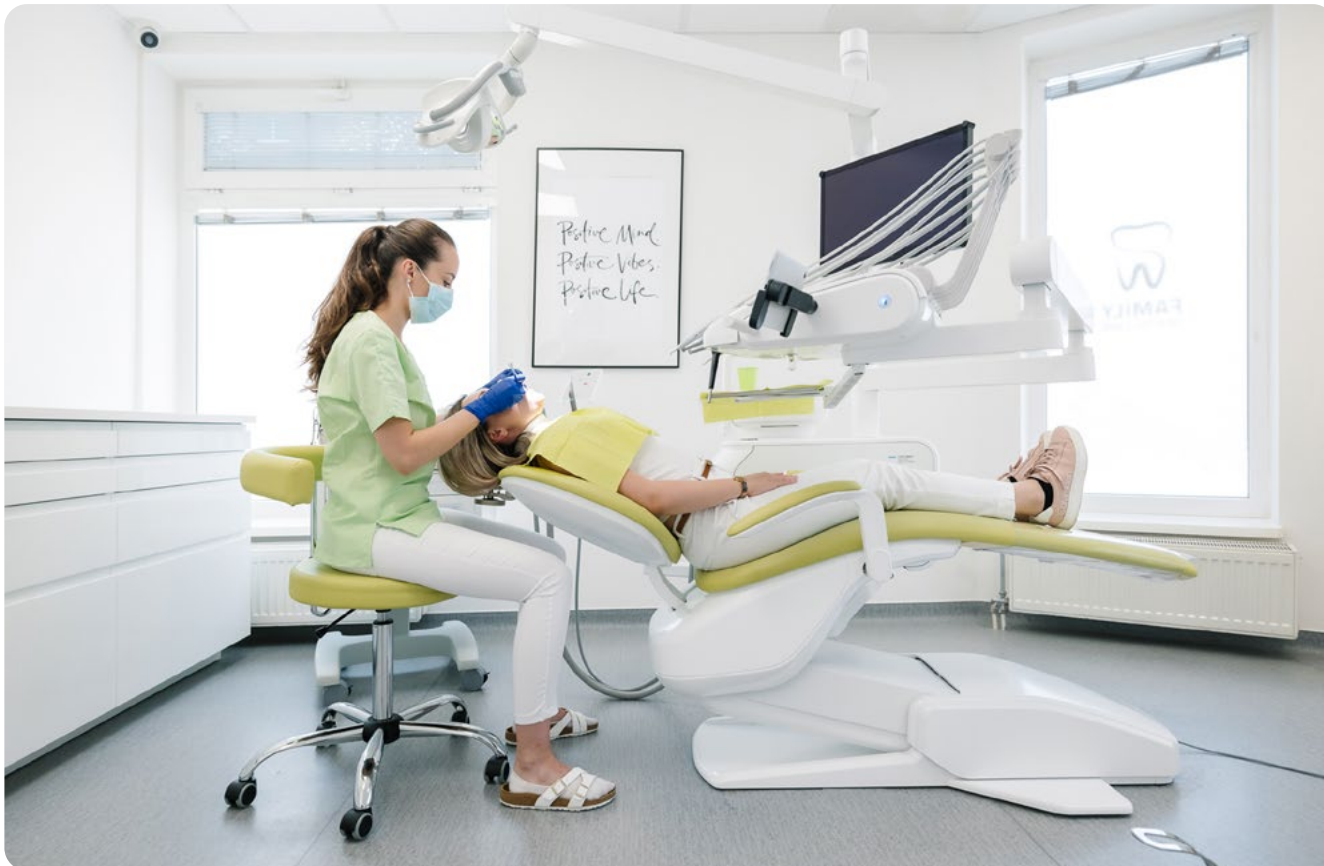

Return on investment.

The economic benefits of smart dental units cannot be overlooked. By optimizing workflows and reducing chairside time, you can treat more patients in a day without compromising the quality of care. This increase in patient throughput translates to improved practice revenues and a higher return on investment.

Marketing and patient appeal.

Model Pro attracts tech-savvy patients. A dental practice equipped with the first smart dental unit is likely to pique the interest of potential patients who appreciate modern, state-of-the-art amenities.



Smart expert support.

I appreciate their minimalist approach. There is a lot more space around the chair and it can be adjusted to even very low positions. This is great because we need to get quite low sometimes. It certainly helps the ergonomics of the work itself. We often work behind the patient's back, and this way I can keep my back completely straight even when I work with the microscope. I can reach everything comfortably and that's the way it should be.

Dr. Martin Lisý,
owner of Stomadent dental clinic




Get to know Diplomat Dental.

Diplomat Dental is a family company with a tradition dating back to 1945. Located in the European dental valley, in Piešťany, approximately 100 kilometers away from Vienna, it holds the perfect position not only in terms of location but also experience gained over the 60 years of development. With thousands of satisfied dentists all over the world, the company is now making the breakthrough into digitized dentistry with the help of the 3rd generation of Nerád family CEO.

Smart manufacturing.

All our products are carefully designed and engineered to meet the specific needs of our customers while always attaining our high quality-standards.

After this process, our beta tester dentists work with the prototypes for up to one year while we use their feedback to fine-tune the product.

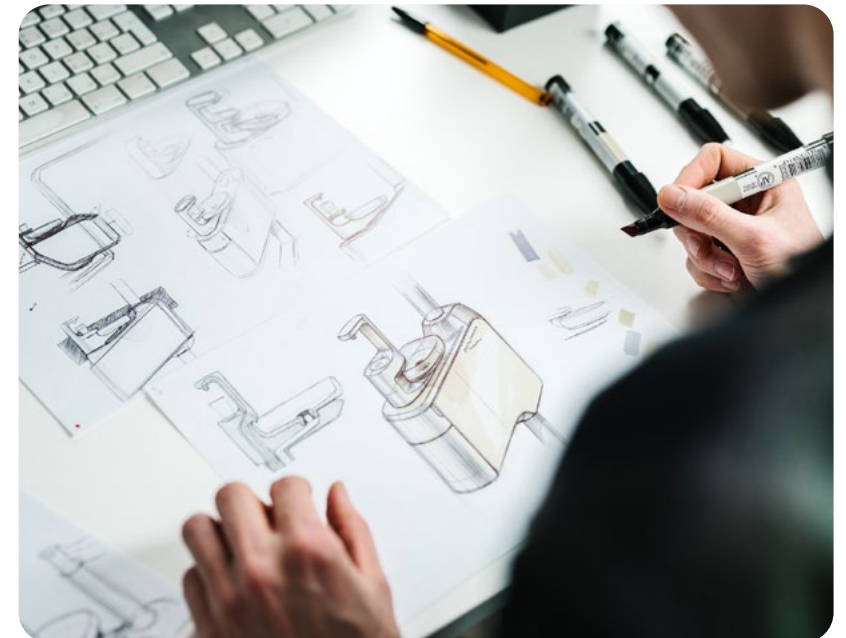

Smart management.

CEO Tomáš Nerád heavily invests into research and development, innovation and new premium quality products. Diplomat Dental under his rule transformed into a global company with state-of-the-art products and smart innovations never before seen on the dental market.


His efforts were rewarded with the most prestigious entrepreneurial award – EY Technological Entrepreneur of the year 2023.




"Our product philosophy is into focus on open platforms, modular and intelligent solutions that improve the workflow of your dental office – making advanced technologies and smart tools easy to use and accessible to every dentist."

Tomas Nerad,
CEO – Diplomat Dental


Dentist element

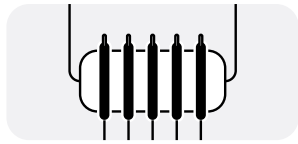


Traditional style
5 instrument option

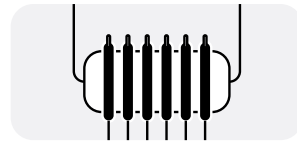

Traditional style
6 instrument option

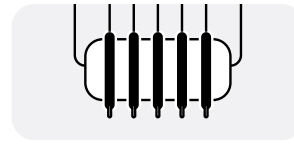

Continental style
5 instrument option

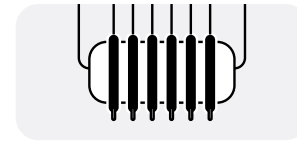

Continental style
6 instrument option

Control pannel



Diplomat Connect app
in iOS



Diplomat Connect app
in Android

Assistant's element



Standard
Basic



Optional
Advanced



Standard
Rotary assistant arm



Optional
Pantographic assistant arm

Cuspidor



Standard
Manual rotary cuspidor



Optional
Automatic rotary cuspidor

Headrest



Standard
**2D Mechanical headrest
Forward/Backward**



Optional
**3D Mechanical headrest
Forward/Backward
and side (Left/Right)
movement**



Optional
**3D Pneumatic headrest
Forward/Backward and side (Left/
Right) movement controlled by
pneumatic brake for one-hand-
control**

Lamp



Standard
XENOS



Optional
MAIA



Optional
MAIA

Footcontroller



Standard
Nok



Optional
Swing

Monitor



Optional
Neovo monitor 22" DR22 white Widescreen
Neovo monitor 22" RX22G black, FULL HD Widescreen

Tray

See more options at page 49




Optional
Extra tray table
sized 370x290 mm on the light pole



Optional
Additional tray table below dentist element
only for TS style


Upholstery types

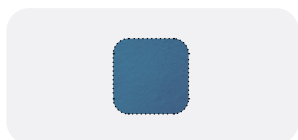

Standard (seamless)
Upholstery



Stitched
Upholstery

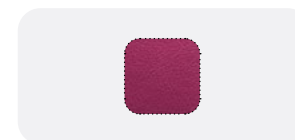

Stitched (Premium)
Upholstery with Active foam

Water unit



Standard
Big and Small suction hoses with selection valve (two filters)

Stool



Optional
With a backrest for the dentist,
With a backrest for the assistant,
Horse-seat for a microscope

Accessories



Optional
Pillow for children



Optional
Protection cushions

Paint



Optional
Complete paint in semi-matt white

Choose your own features.

Configure Model Pro to become your dream dental unit.

Colour version of dental unit	
Colored plastic cover on water unit	<input type="radio"/>
Complete unit painted in semi-matt white	<input type="radio"/>
Complete unit painted in customized color	
Chair	
CHAIR UPGRADES	
Chair version with synchronization of seat and backrest	<input type="radio"/>
Front panel for external media connections (air, water, power)	<input type="radio"/>
Folding knee chair with automatically extending footrest (available only with Model Pro 600 and Model Pro 800)	<input type="radio"/>
HEADRESTS	
2D headrest	<input type="radio"/>
Mechanical headrest swinging 3D	<input type="radio"/>
Pneumatic headrest swinging 3D	<input type="radio"/>
HANDRESTS	
Left hand rest	<input type="radio"/>
Right hand rest	<input type="radio"/>
UPHOLSTERY	
Standard (seamless) upholstery (Standard, Special, Special Plus colors)	<input type="radio"/>
Stitched upholstery (Standard, Special, Special Plus colors)	<input type="radio"/>
Premium Stitched upholstery with ACTIVE FOAM	<input type="radio"/>
UPHOLSTERY COLOR	
SPECIAL or SPECIAL PLUS colors surcharge	<input type="radio"/>
<i>Please write down upholstery color code</i>	
ACCESSORIES	
Pillow for children	<input type="radio"/>
Protection cushion	<input type="radio"/>
Dentist element	
DELIVERY SYSTEM	
Lower delivery – traditional	<input type="radio"/>
Upper delivery – continental	<input type="radio"/>
DENTIST ARM	
Without pneumatic brake (only mechanical brake)	<input type="radio"/>
With pneumatic brake	<input type="radio"/>

**INSTRUMENTS**

5 instruments	<input type="radio"/>
6 instruments	<input type="radio"/>

SYRINGES

Minimate 3F syringe straight – INOX (metal), round-shaped	<input type="radio"/>
Minimate 3F syringe angled – INOX (metal), round-shaped	<input type="radio"/>
Minilight 3F syringe straight – INOX (metal), square-shaped	<input type="radio"/>
Minilight 3F syringe angled – INOX (metal), square-shaped	<input type="radio"/>
Minilight 6F syringe straight – INOX (metal), square-shaped	<input type="radio"/>
Minilight 6F syringe angled – INOX (metal), square-shaped	<input type="radio"/>
Minilight 6F syringe straight with light and module – INOX (metal), square-shaped	<input type="radio"/>

TURBINE HOSE

Midwest fibre-optic hose for a turbine with accessories	<input type="radio"/>
Turbine output regulation (for one or two turbines)	<input type="radio"/>

DENSIM DX MICROMOTORS

DENSIM DX motor with accessories	<input type="radio"/> Number of micromotors ____
DENSIM DX BLUE motor with accessories	<input type="radio"/> Number of micromotors ____
SET DENSIM DX PRO motor with accessories	<input type="radio"/> Number of micromotors ____
SET DENSIM DX PRO BLUE motor with accessories	<input type="radio"/> Number of micromotors ____

SURGICAL MICROMOTOR OPTIONS

Peristaltic Pump and EXTRA DENSIM SRG LED surgical motor with accessories, possible to combine with DX PRO or DX PRO BLUE motor, RPM range 100-40.000, LED light, T=5,0N cm (motor as last instrument on dentist element from the left)	<input type="radio"/>
---	-----------------------

SCALERS

LM-ProPower UltraLED scaler with light with module (4 tips) (Amdent)	<input type="radio"/>
EMS MiniPiezon FS-433 scaler without light (electric regulation of performance) (3 tips)	<input type="radio"/>
EMS MiniPiezon FS-432 LED scaler with light (electric regulation of performance) (3 tips)	<input type="radio"/>
EMS MiniPiezon FS-367 LED scaler with light (electric regulation of performance) (3 tips)	<input type="radio"/>
Satelec SP Xinetec scaler without light (1 tip)	<input type="radio"/>
Satelec NEWTRON scaler without light (3 tips)	<input type="radio"/>
Satelec NEWTRON LED scaler with light (3 tips)	<input type="radio"/>
NSK VA170 scaler without light with module (3 tips)	<input type="radio"/>
NSK VA170 LED – Lux scaler with light with module (3 tips)	<input type="radio"/>
Woodpecker UDS-N3 LED scaler (6 tips)	<input type="radio"/>


CURING LIGHTS

Mectron Starlight	<input type="radio"/>
Monitex curing light	<input type="radio"/>

Others	
Heating of cooling water for instruments	<input type="radio"/>
Whip lock only for CS (upper hose) delivery system	<input type="radio"/>
Water unit	
SUCTION SYSTEMS	
WET SUCTION	
Wet suction with multi-station valve (price included in basic configuration)	<input type="radio"/>
Wet suction with multi-station valve + Dürr cuspidor valve	<input type="radio"/>
DRY SUCTION	
Cattani miniseparator (price included in basic configuration)	<input type="radio"/>
METASYS ECO-L1, separator water-air	<input type="radio"/>
Dürr CS1 separator	<input type="radio"/>
MST1 amalgam separator (including sample disinfection set + applicator + replacement container) separation rate 98,6%	<input type="radio"/>
Dürr CAS-1 amalgam separator, (including replacement container) separation rate 97,9%	<input type="radio"/>
HYGIENE	
automatic disinfection (of cup filling and cooling water)	<input type="radio"/>
automatic decontamination (of suction hoses and bowl rinsing)	<input type="radio"/>
Integrated WEK System in compliance of EN1717	<input type="radio"/>
Electronic bowl rotation	<input type="radio"/>
Boiler (for water heating)	<input type="radio"/>
Central water supply	<input type="radio"/>
Assistant element	
ASSISTANT ARM	
Pantographic (self-balancing) assistant's arm, mechanic brake, six positions holder with big and small Cattani suction aspirators, foil keyboard regulating cup filling, bowl rinsing, and chair	<input type="radio"/>
INSTRUMENTS	
SYRINGES	
Minimate 3F syringe straight – INOX (metal), round-shaped	<input type="radio"/>
Minimate 3F syringe angled – INOX (metal), round-shaped	<input type="radio"/>
Minilight 3F syringe straight – INOX (metal), square-shaped	<input type="radio"/>
Minilight 3F syringe angled – INOX (metal), square-shaped	<input type="radio"/>
Special accessories for assistant's syringe if WEK system or disinfection system is installed	<input type="radio"/>
OTHERS	
Additional saliva ejector (not possible to combine with any amalgam separator)	<input type="radio"/>
Support of big and small suction hoses (2 pcs. of roller)	<input type="radio"/>
Diplomat intraoral USB camera with software for connection with PC and accessories (10 m active USB cable, software licence key, and 50 pcs. of protection sleeves included)	<input type="radio"/>

Dental lights	
Faro MAIA LED lamp with sensor	<input type="radio"/>
Sirius Essential (35.000 Lux)	<input type="radio"/>
Sirius Plus (35.000 Lux)	<input type="radio"/>
Sirius Ultra (50.000 Lux)	<input type="radio"/>
Without the integration of a dental pantograph and dental light, a small cover	<input type="radio"/>
Tray tables and integrations	
Adaptation for monitor	<input type="radio"/>
Neovo monitor 22" DR22 white (wide screen)	<input type="radio"/>
Monitor Neovo 22" RX22G black, FULL HD – Widescreen	<input type="radio"/>
Extra tray table for light pole sized 370x290 mm	<input type="radio"/>
Tissue Holder	<input type="radio"/>
Tray	
Bigger Tray size 455x325 mm (for both TS & CS delivery system, all models)	<input type="radio"/>
Additional tray table below dentist element (only model pro TS style)	<input type="radio"/>
Left 380x285 mm (for TS style, all models)	<input type="radio"/>
Double 2x 380x285 mm (for TS style, all models)	<input type="radio"/>
Foot controllers	
NOK wired	<input type="radio"/>
NOK wireless	<input type="radio"/>
SWING wired	<input type="radio"/>
SWING wireless	<input type="radio"/>
Integrations with intraoral imaging systems	
Adapter on the light pole for intraoral X-ray units	<input type="radio"/>
NOK wireless	<input type="radio"/>
Second USB Module to connect Digital Sensor on the Dentist Element	<input type="radio"/>
Dental stools	
TYPE	
D10L with back-rest for the dentist	<input type="radio"/>
D10L with back-rest for assistant	<input type="radio"/>
D10L horse-seat	<input type="radio"/>
D10L for microscope	<input type="radio"/>
ACCESSORIES	
Leg support with lock ring	<input type="radio"/>

Model Pro 500

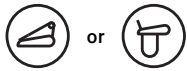

Optional upper or lower hose delivery system

Convertible
(For both right- and left-handed)

Carried version

5 instruments on the dentist's table

—

—

Optional wireless multifunctional foot control


—

Customizable colours on the seat upholstery

Customizable colours on spittoon block

—

Model Pro 600

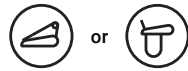

Optional upper or lower hose delivery system

—

Lift version

5 instruments on the dentist's table

—

—

Optional wireless multifunctional foot control

—

Customizable colours on the seat upholstery

Customizable colours on spittoon block

Possible to equip with X-ray (extra arm)

Model Pro 700



Optional upper or lower hose delivery system

Convertible
(For both right- and left-handed)

Carried version

6 instruments on the dentist's table

Synchronization of seat & backrest *

Electronic bowl rotation *

Optional wireless multifunctional foot control

Complete unit painted in semi-matt white *

Customizable colours on the seat upholstery

Customizable colours on spittoon block

—

Model Pro 800



Optional upper or lower hose delivery system

—

Lift version

6 instruments on the dentist's table

Synchronization of seat & backrest *

Electronic bowl rotation *


Optional wireless multifunctional foot control

Complete unit painted in semi-matt white *

Customizable colours on the seat upholstery

Customizable colours on spittoon block

Possible to equip with X-ray (extra arm)

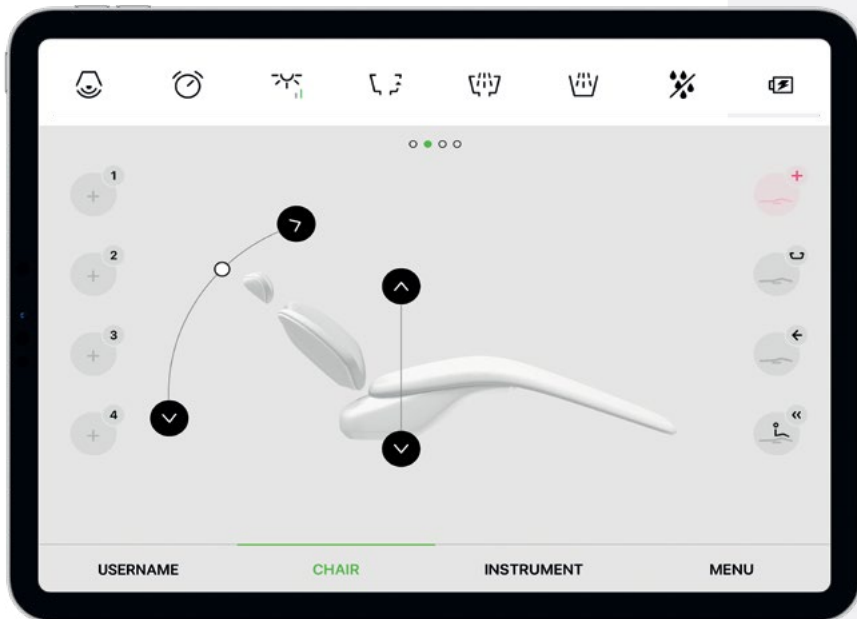

Connect with our sales team:

+421 33 7954 111

info@diplomat-dental.com

Follow us:

@diplomat_dental

Diplomat dental

Diplomat Dental Solutions

Diplomat Dental Solutions



www.diplomat-dental.com

Load & Play



Digitise your practice through a simple tablet and get the most advanced and user-friendly control system for dental units available in the market.

Use your own tablet to control your dental unit and instruments. Both Android and iOS are fully supported. Data backup on iCloud or Android drive. Quick and secure access to patient data, X-rays, photos. Automatic updates, unlimited amount of users and a friendly interface.

1

Download the App
Diplomat Connect.

2

Activate Bluetooth.

3

Ready!
Your dental unit is ready to take off.



"Diplomat Connect" is available for iPad on the App Store or Android App on Google play.

